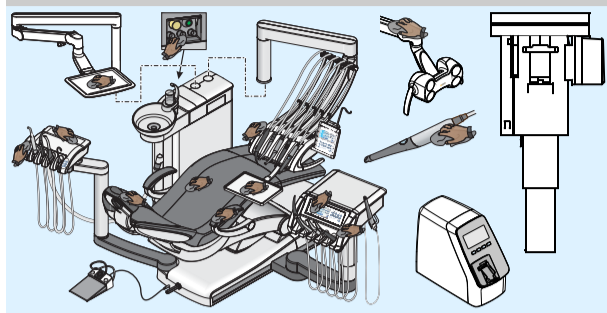
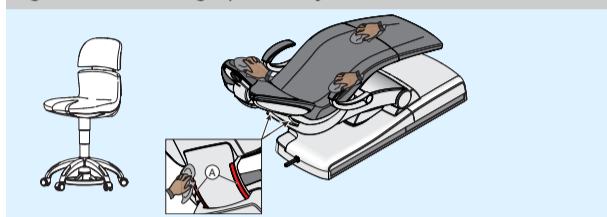
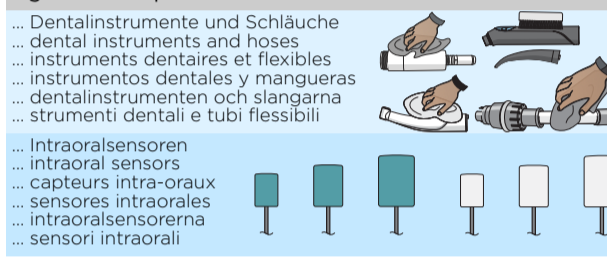
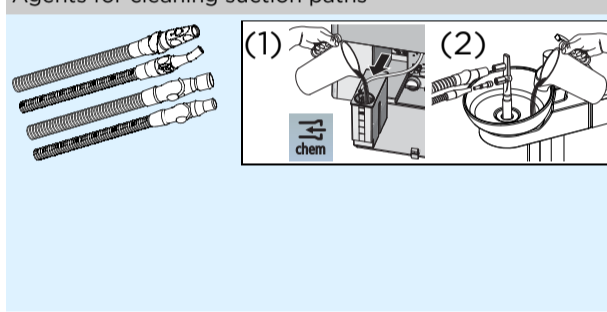
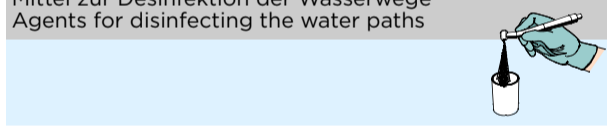
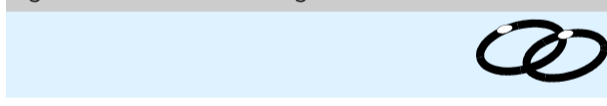
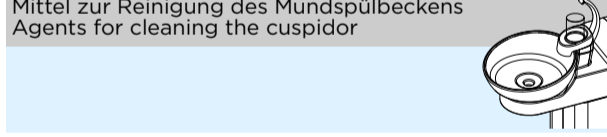
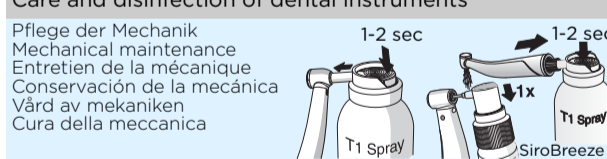
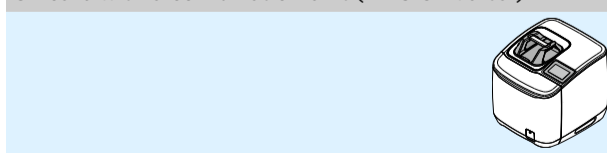
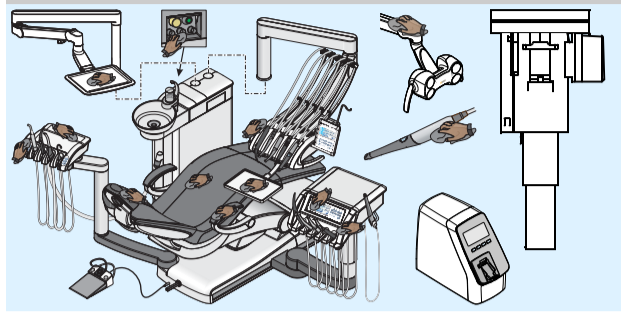
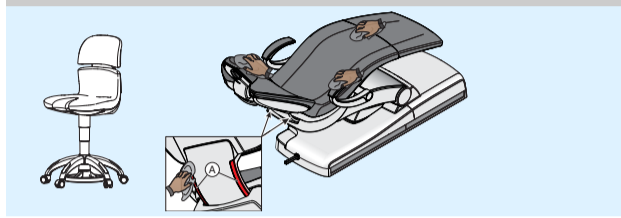
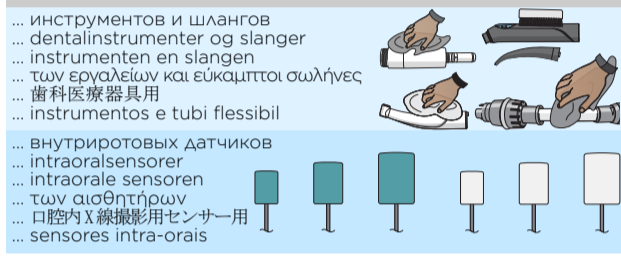
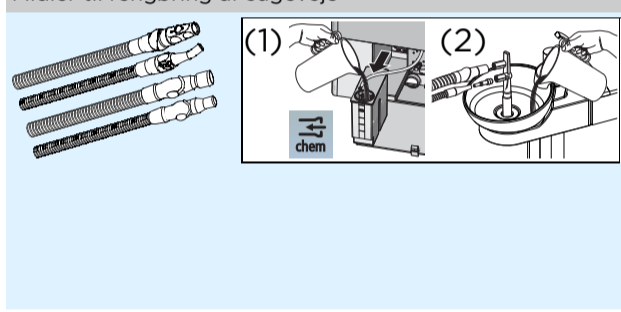
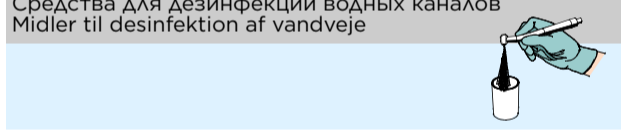
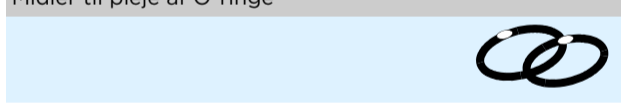
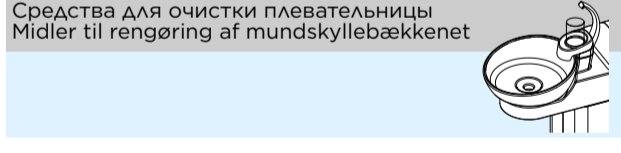

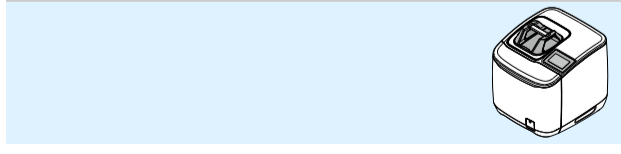



<p>Oberflächendesinfektionsmittel (auch für Polster) Surface disinfectants (also for upholstery)</p> 	<p>Produits de désinfection de surface (également pour coussins) Desinfectante de superficies (también para acolchado)</p> <p>Alpro</p> <ul style="list-style-type: none"> PlastiSept eco JumboWipes (A) <p>Dürr</p> <ul style="list-style-type: none"> FD 366 sensitive FD 366 sensitive wipes <p>Henry Schein</p> <ul style="list-style-type: none"> Maxima Non-Alcoholic Surface Disinfection <p>(A) Freigabe gilt nur für Behandlungseinheiten / release applies only to treatment centres / autorisation valable seulement pour unités de traitement / versión válida solo para unidades de tratamiento / frígivning gäller endast för behandlingsenheter / il rilascio si applica unicamente ai riuniti</p>	<p>Desinfektionsmedel för ytor (även för dynor) Disinfettante per le superfici (includo le imbottiture)</p> <p>Pluradent</p> <ul style="list-style-type: none"> Plurawipes Wet Sensitiv <p>Schülke & Mayr</p> <ul style="list-style-type: none"> Mikrozyd sensitive Liquid Mikrozyd sensitive Wipes 	
<p>Mittel zur Reinigung der Polster Agent for cleaning upholstery</p> 	<p>Produit pour le nettoyage des coussins Producto de limpieza de los acolchados</p> <p>Dürr</p> <ul style="list-style-type: none"> FD 360 <p>Reinigungsschwamm nicht für Lounge-Polster verwenden! Do not use cleaning sponge on lounge upholstery! Ne pas utiliser l'éponge de nettoyage sur la sellerie Lounge. ¡No use la esponja de limpieza para acolchados lounge! Använd inte rengöringssvamp för Lounge-dynor! Non utilizzare la spugna di pulizia per le imbottiture lounge!</p>	<p>Medel för rengöring av dynor Prodotto per la pulizia delle imbottiture</p>	
<p>Mittel zur Wischdesinfektion der ... Agents for wipe disinfection of the ...</p> <p>... Dentalinstrumente und Schläuche ... dental instruments and hoses ... instruments dentaires et flexibles ... instrumentos dentales y mangueras ... dentalinstrumenten och slangarna ... strumenti dentali e tubi flessibili</p> <p>... Intraoralsensoren ... intraoral sensors ... capteurs intra-oraux ... sensores intraorales ... intraoralsensorerna ... sensori intraorali</p> 	<p>Produits de désinfection par essuyage pour ... Desinfectantes por frotamiento de ...</p> <p>Alpro</p> <ul style="list-style-type: none"> MinutenSpray-classic MinutenWipes <p>Kerr Corporation</p> <ul style="list-style-type: none"> Original CaviCide Original CaviWipes <p>Alpro</p> <ul style="list-style-type: none"> PlastiSept PlastiSept-Wipes <p>Kerr Corporation</p> <ul style="list-style-type: none"> Original CaviCide Original CaviWipes 	<p>Medel för avtorkningsdesinfektion av ... Prodotto per la disinfezione con panno umido di ...</p> <p>Reckitt Benckiser</p> <ul style="list-style-type: none"> Lysol <p>Schülke & Mayr</p> <ul style="list-style-type: none"> Mikrozyd sensitive Liquid Mikrozyd sensitive Wipes 	
<p>Mittel zur Reinigung der Saugwege Agents for cleaning suction paths</p> 	<p>Produits de nettoyage des canaux d'aspiration Productos de limpieza para conductos de aspiración</p> <p>Alpro</p> <ul style="list-style-type: none"> AlproJet-D (1+2) AlproJet-DD (1+2) AlproJet-W (2) <p>DE Healthcare Products</p> <ul style="list-style-type: none"> Safe R Evac liquid (1+2) <p>Dürr</p> <ul style="list-style-type: none"> Orotol Ultra (2) MD 555 (2) Orotol Plus (1+2) <p>Henry Schein</p> <ul style="list-style-type: none"> EuroSept Max Evac System Liquid (1+2) <p>Metasys</p> <ul style="list-style-type: none"> GREEN&CLEAN M2 (2) <p>Merz Hygiene GmbH</p> <ul style="list-style-type: none"> Mucadont-AS (2) 	<p>Medel för rengöring av sugvägar Prodotto per la pulizia delle condotte di aspirazione</p> <p>Omnident</p> <ul style="list-style-type: none"> Omnisuc (1+2) <p>Orbis Dental</p> <ul style="list-style-type: none"> ORBI-Sept Absauganlagen-desinfektion (1+2) <p>Pluradent</p> <ul style="list-style-type: none"> Pluraseptol (1+2) <p>Prisman</p> <ul style="list-style-type: none"> SunSept Absauganlagen-desinfektion (1+2) <p>Schülke & Mayr</p> <ul style="list-style-type: none"> Aspirmatic (1+2) Aspirmatic Cleaner (2) 	
<p>Mittel zur Desinfektion der Wasserwege Agents for disinfecting the water paths</p> 	<p>Produits de désinfection des circuits d'eau Desinfectante para los conductos de agua</p> <p>Dentsply Sirona</p> <ul style="list-style-type: none"> Dentosept S* Dentosept P* <p>* Nicht in allen Ländern verfügbar / not available in all countries / disponibilité limitée à certains pays / no disponible en todos los países / ej tillgänglig i alla länder / disponibili non in tutti i paesi</p>		<p>Desinficeringsmedel för vattenkanalerna Prodotto per la disinfezione del circuito idrico</p>
<p>Mittel zur Pflege der O-Ringe Agent for care of the O-rings</p> 	<p>Produits d'entretien des joints toriques Productos para la conservación de los anillos toroidales</p> <p>Dentsply Sirona</p> <ul style="list-style-type: none"> Unisilikon LCA 3801 LUBE "O" Seal 		<p>Medel för vård av O-ringar Prodotto per la cura degli O-ring</p>
<p>Mittel zur Reinigung des Mundspülbeckens Agents for cleaning the cuspidor</p> 	<p>Produit de nettoyage de la cuvette de crachoir Producto para limpiar la escupidera</p> <p>Alpro</p> <ul style="list-style-type: none"> Alpro Cleaner <p>Dürr</p> <ul style="list-style-type: none"> MD 550 		<p>Rengöringsmedel till spottkoppen Prodotto per la pulizia della bacinella</p>
<p>Pflege und Desinfektion von Dentalinstrumenten Care and disinfection of dental instruments</p> <p>Pflege der Mechanik Mechanical maintenance Entretien de la mécanique Conservación de la mecánica Vård av mekaniken Cura della meccanica</p>  <p>Reinigung und Desinfektion der Medienkanäle* Cleaning and disinfection of media channels* Nettoyage et désinfection des circuits de fluides* Limpieza y desinfección de los canales de medios* Rengöring och desinfektion av mediekkanaler* Pulizia e disinfezione dei canali dei liquidi*</p> <p>* nicht für Getriebe und Treibluftkanal / not for gears and impelling air channel / pas pour le réducteur ni le circuit d'air propulseur / excluidos accionamientos y canal de aire de accionamiento / inte för drev och drivluftkanal / non per ingranaggio e canale dell'aria compressa</p>	<p>Entretien et désinfection d'instruments dentaires Conservación y desinfección de instrumentos dentales</p> <p>Dentsply Sirona</p> <ul style="list-style-type: none"> T1 Spray SiroBreeze <p>Alpro</p> <ul style="list-style-type: none"> WL-clean WL-cid WL-dry <p>Metasys</p> <ul style="list-style-type: none"> GREEN&CLEAN IK 		<p>Vård och desinfektion av dentalinstrumenten Cura e disinfezione degli strumenti dentali</p>
<p>Ölpflege mit einem Kombinationsgerät (DAC Universal) Oil care with a combination unit (DAC Universal)</p> 	<p>Lubrification avec un appareil combiné (DAC Universal) Lubrificación con equipo combinado (DAC Universal)</p> <p>Dentsply Sirona</p> <ul style="list-style-type: none"> Nitram Oil Nitram Oil 2 		<p>Oljevård med ett kombinationsinstrument (DAC Universal) Lubrificazione con un apparecchio combinato (DAC Universal)</p>
<p>Download → dentsplysirona.com/manuals</p>			<p>Anwendung, siehe Gebrauchsanweisung Eingetragene Warenzeichen sind nicht gekennzeichnet!</p> <p>For use, refer to the operating instructions Registered trademarks are not marked!</p> <p>Application, se reporter à la notice d'utilisation Les marques déposées ne sont pas identifiées !</p> <p>Aplicación, véanse las instrucciones de uso ¡Las marcas registradas no están señaladas!</p> <p>Användning, se bruksanvisning Inregistrerade varumärken är ej märkta!</p> <p>Per l'applicazione vedere le Istruzioni per l'uso I marchi registrati non sono contrassegnati!</p>

<p>Средство дезинфекции поверхности (в т.ч. мягкой мебели) Overfladedesinfektionsmiddel (også til polstre)</p> 	<p>Oppervlaktedesinfectiemiddel (ook voor bekleding) Απολυμαντικό επιφανειών (και για επενδύσεις)</p> <p>Alpro</p> <ul style="list-style-type: none"> PlastiSept eco JumboWipes (A) <p>Dürr</p> <ul style="list-style-type: none"> FD 366 sensitive FD 366 sensitive wipes <p>Henry Schein</p> <ul style="list-style-type: none"> Maxima Non-Alcoholic Surface Disinfection <p>(A) Разрешение касается только стоматологических установок / godkendelsen gælder kun for behandlingsenheder / vrijgave geldt alleen voor de behandelingsseenheden / η έγκριση ισχύει μόνο για μονάδες θεραπειάς / トリートメントセンター用にのみリリースが適用されます / a autorização se aplica apenas unidades de tratamento dentário</p>	<p>環境表面消毒剤 (シート用) Desinfetante de superfícies (também para estofos)</p> <p>Pluradent</p> <ul style="list-style-type: none"> Plurawipes Wet Sensitiv <p>Schülke & Mayr</p> <ul style="list-style-type: none"> Mikrozid sensitive Liquid Mikrozid sensitive Wipes
<p>Средства для очистки мягкой обивки Midler til rengøring af polster</p> 	<p>Middelen voor de reiniging van kussens Μέσα καθαρισμού επενδύσεων</p> <p>Dürr</p> <ul style="list-style-type: none"> FD 360 <p>Не использовать губку для обивки дивана! Brug ikke en rengøringssvamp til Lounge-polsteret! Reinigings spons niet voor Lounge-kussens gebruiken! Μη χρησιμοποιείτε το σφουγγάρι καθαρισμού για επενδύσεις lounge! 清掃用のスポンジは、外来待合室のソファには、使用しないでください! Não usar a esponja de limpeza para os estofos Lounge!</p>	<p>チェア清掃剤 Produto de limpeza dos estofos</p>
<p>Средства для дезинфекции протиранием ... Midler til viskedesinfektion af ...</p> <p>... инструментов и шлангов ... dentalinstrumenter og slanger ... instrumenten en slangen ... των εργαλείων και εύκαμπτοι σωλήνες ... 歯科医療器具用 ... instrumentos e tubi flessibil</p> <p>... внутриротовых датчиков ... intraoralsensorer ... intraorale sensoren ... των αισθητήρων ... 口腔内 X線撮影用センサー用 ... sensores intra-orais</p> 	<p>Middel voor veegdesinfectie van... Μέσα για την απολύμανση με σκούπισμα ...</p> <p>Alpro</p> <ul style="list-style-type: none"> MinutenSpray-classic MinutenWipes <p>Kerr Corporation</p> <ul style="list-style-type: none"> Original CaviCide Original CaviWipes <p>Alpro</p> <ul style="list-style-type: none"> PlastiSept PlastiSept-Wipes <p>Kerr Corporation</p> <ul style="list-style-type: none"> Original CaviCide Original CaviWipes 	<p>清拭消毒用製品 Produto para limpeza por fricção de ...</p> <p>Reckitt Benckiser</p> <ul style="list-style-type: none"> Lysol <p>Schülke & Mayr</p> <ul style="list-style-type: none"> Mikrozid sensitive Liquid Mikrozid sensitive Wipes
<p>Средства для очистки каналов всасывания Midler til rengøring af sugeveje</p> 	<p>Middel voor de reiniging van de zuigwegen Μέσα για τον καθαρισμό των οδών αναρρόφησης</p> <p>Alpro</p> <ul style="list-style-type: none"> AlproJet-D (1+2) AlproJet-DD (1+2) AlproJet-W (2) <p>DE Healthcare Products</p> <ul style="list-style-type: none"> Safe R Evac liquid (1+2) <p>Dürr</p> <ul style="list-style-type: none"> Orotol Ultra (2) MD 555 (2) Orotol Plus (1+2) <p>Henry Schein</p> <ul style="list-style-type: none"> EuroSept Max Evac System Liquid (1+2) <p>Metasys</p> <ul style="list-style-type: none"> GREEN&CLEAN M2 (2) <p>Merz Hygiene GmbH</p> <ul style="list-style-type: none"> Mucadont-AS (2) 	<p>バキューム回路洗浄剤 Produto para limpeza das vias de aspiração</p> <p>Omnident</p> <ul style="list-style-type: none"> Omnisuc (1+2) <p>Orbis Dental</p> <ul style="list-style-type: none"> ORBI-Sept Absauganlagen-desinfektion (1+2) <p>Pluradent</p> <ul style="list-style-type: none"> Pluraseptol (1+2) <p>Prisman</p> <ul style="list-style-type: none"> SunSept Absauganlagen-desinfektion (1+2) <p>Schülke & Mayr</p> <ul style="list-style-type: none"> Aspirmatic (1+2) Aspirmatic Cleaner (2)
<p>Средства для дезинфекции водных каналов Midler til desinfektion af vandveje</p> 	<p>Middel voor de desinfectie van de waterwegen Μέσα για την απολύμανση των διόδων νερού</p> <p>Dentsply Sirona</p> <ul style="list-style-type: none"> Dentosept S* Dentosept P* <p>* доступны не во всех странах / fås ikke i alle lande / niet in alle landen leverbaar / δεν είναι διαθέσιμος σε όλες τις χώρες / すべての国で利用できるわけではありません / não disponível em todos os países</p>	<p>水管路洗浄剤 Produto para desinfeção dos canais de água</p>
<p>Средство для ухода за уплотнительными кольцами Midler til pleje af O-ringe</p> 	<p>Middel voor de reiniging van de o-ringen Μέσα για τη φροντίδα των δακτυλίων O</p> <p>Dentsply Sirona</p> <ul style="list-style-type: none"> Unisilikon LCA 3801 LUBE "O" Seal 	<p>O リングメンテナンス用液剤 Produto para tratamento dos o-rings</p>
<p>Средства для очистки плевательницы Midler til rengøring af mundskyllebækkenet</p> 	<p>Middel voor de reiniging van de spoelbak Μέσα για τον καθαρισμό του πτυελοδοχείου</p> <p>Alpro</p> <ul style="list-style-type: none"> Alpro Cleaner <p>Dürr</p> <ul style="list-style-type: none"> MD 550 	<p>ベースン洗浄剤 Produto para limpeza da cuspeira</p>
<p>Уход и дезинфекция стоматологических инструментов Pleje og desinfektion af dentalinstrumenter</p> <p>Уход за механическими компонентами 1-2 sec Pleje af mekanik 1-2 sec Verzorging van de mechanica 1-2 sec Φροντίδα των μηχανικών στοιχείων 1-2 sec 機械動作部のメンテナンス 1-2 sec Tratamento do sistema mecânico 1-2 sec</p>  <p>Очистка и дезинфекция каналов рабочих сред* Rengøring og desinfektion af mediekkanaler* Reiniging en desinfectie van de mediakanalen* Καθαρισμός και απολύμανση των καναλιών μέσων* 管路*の洗淨/消毒* Limpeza e desinfeção dos canais dos produtos*</p> <p>* не только для редуктора и канала рабочего воздуха / ikke til drev og drivluftskanal / niet voor transmissies en persluchtkanaal / όχι για μηχανισμούς κίνησης και κανάλι φέροντος αερίου / ギヤ部、圧縮空気の配管には、使用できません / não para engrenagem e canal de ar de propulsão</p>	<p>Verzorging en desinfectie van tandheelkundige instrumenten Φροντίδα και απολύμανση οδοντικών εργαλείων</p> <p>Dentsply Sirona</p> <ul style="list-style-type: none"> T1 Spray SiroBreeze <p>Alpro</p> <ul style="list-style-type: none"> WL-clean WL-cid WL-dry <p>Metasys</p> <ul style="list-style-type: none"> GREEN&CLEAN IK 	<p>歯科医療器具のメンテナンス/消毒 Tratamento e desinfeção de instrumentos de tratamento dentário</p>
<p>Смазывание с помощью комбинированного устройства (DAC Universal) Oliepleje med et kombinationsapparat (DAC Universal)</p> 	<p>Oliebehandeling met een combinatie-apparaat (DAC Universal) Περιποίηση με λάδι με μια συνδυαστική συσκευή (DAC Universal)</p> <p>Dentsply Sirona</p> <ul style="list-style-type: none"> Nitram Oil Nitram Oil 2 	<p>組み合わせ装置によるオイルケア (DACユニバーサル) Lubrificação a óleo com um aparelho combinado (DAC Universal)</p>
<p>Download → dentsplysirona.com/manuals</p> <p>Использование - см. Руководство по эксплуатации Зарегистрированные товарные знаки не помечены!</p> <p>Anvendelse, se brugsanvisningen Registrerede varemærker er ikke markerede!</p>	<p>Gebruik, zie gebruikershandleiding Geregistreerde handelsmerken zijn niet gekentekend!</p> <p>Εφαρμογή, βλέπε οδηγίες χρήσης Τα σήματα κατατεθέντα δεν επισημαίνονται!</p>	<p>ご使用にあたっては、取扱説明書をご覧ください。 登録商標のマークは、表示しておりません!</p> <p>Utilização, veja instruções de uso As marcas registradas não estão identificadas!</p>

Care, cleaning and disinfection agents

For USA only! 

Surface disinfectants (also for upholstery)

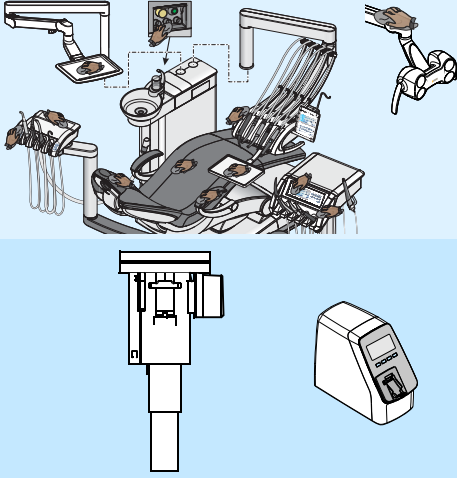

Treatment Centers

Kerr Corporation

- Original CaviCide™
- Original CaviWipes™

Patterson


- pdCARE

all other Dentsply Sirona products

Kerr Corporation


- Original CaviCide™
- Original CaviWipes™

Agent for cleaning upholstery

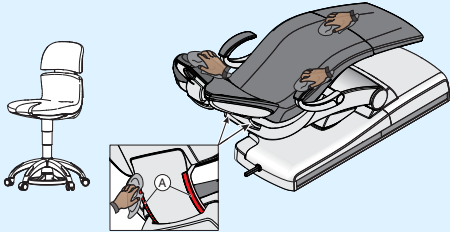

Soap solution based on nonionic and anionic surfactants

Agents for cleaning suction paths

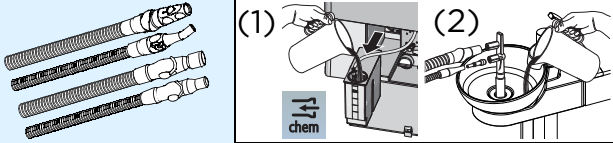


Air Techniques

- Monarch CleanStream (1+2)

Metasys

- GREEN&CLEAN M2 (2)


Agents for wipe disinfection the ...

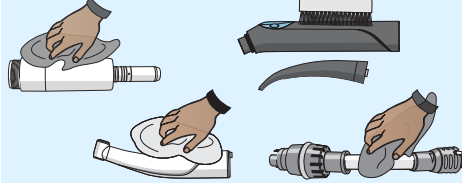

... dental instruments and hoses

Kerr Corporation

- Original CaviCide™
- Original CaviWipes™

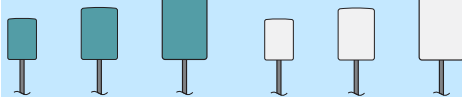

... intraoral sensors

Kerr Corporation

- Original CaviCide™
- Original CaviWipes™


Waterline Treatment Solutions



Dentsply Sirona

- Dentosept P Waterline Treatment Solution

Care and disinfection of dental instruments

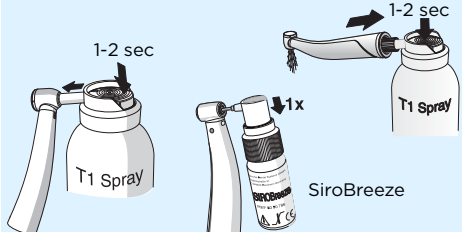

Mechanical maintenance

Dentsply Sirona

- T1 Spray

Agent for care of the O-rings



Dentsply Sirona

- LUBE "O" Seal

Download ⇨ dentsplysirona.com/manuals
 For use, refer to the operating instructions.
 Registered trademarks are not marked!