

Alternative for  
**Reciprocating and Blue Instruments**



**EDGE ONE R UTOPIA™**

**THE COMPLETE SYSTEM**  
YOU WERE LOOKING FOR





PRODUCTS

EDGE ONE R UTOPIA™

### The Remarkable EdgeOne-R Utopia™

The next-generation EdgeOne-R Utopia™ is our most advanced reciprocating file yet. Designed as a complete single-file solution, it simplifies your workflow—you just need one instrument to shape the canal.

We believe that **EdgeOne-R Utopia™** strikes the perfect balance of safety, strength, and cutting efficiency while offering compatibility with standard motors and handpieces. The incorporation of a size 20 file in the system makes it ideal for negotiating even the most challenging canals, including calcified ones, reducing the need for multiple files in your practice.

### Save chair time


The new FireWire Blaze heat treatment process adds improved cutting efficiency to our files while continuing to provide the high flexibility you're accustomed to with our original FireWire heat treatment. These files are engineered for strength and safety, helping you reduce chair time and taking your performance to the next level.

**You will love the feel and performance of this system!**

### Better comfort

Experience improved convenience with our upgraded design! The new **acrylic stopper** is more robust, while enhanced calibration markings make length control easier. Our innovative **Enhanced Handle Fit** provides real-time apex locator readings, and the reduced shank length offers increased posterior access, making every procedure smoother.

### Pre-Sterilized

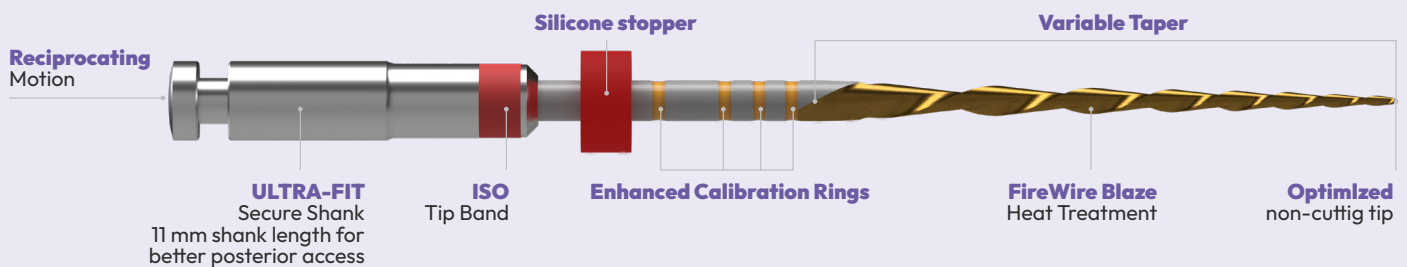

#### S-Shaped Cross Section

Available in 3 lengths: 21, 25, 31 mm  
Improved debris removal




High Cutting Efficiency



Coated Finish



High Strength

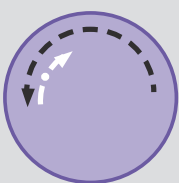


Reciprocating 170° counterclockwise and 60° clockwise.  
Torque: 4.0 N/cm 408 g/cm  
Resulting speed: 400 rpm

Order Now: [EdgeEndo.com](http://EdgeEndo.com)

BEGIN:


FOLLOW WITH:

**GLIDE PATH**

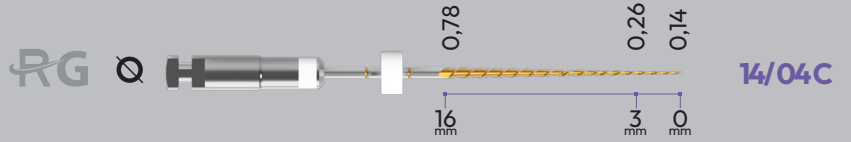


**CALCIFIED CANALS**

**K-File**  
#10 passively

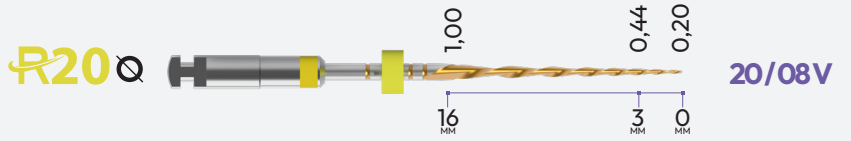


**NARROW CANALS**

**K-File**  
#15 PASSIVELY

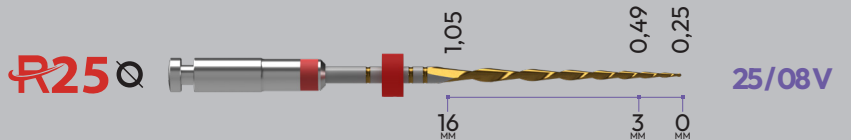


**MEDIUM CANALS**

**K-File**  
#20 PASSIVELY

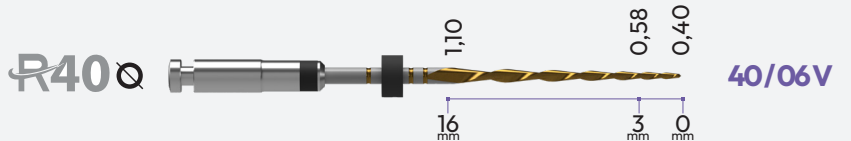

**LARGE CANALS**

**K-File**  
#30 PASSIVELY

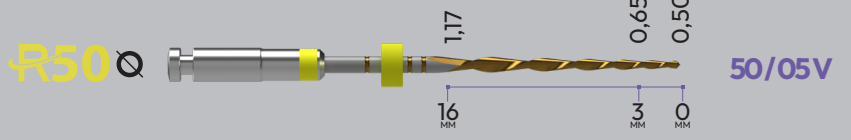

For Glide Path:

6 - Pack STERILE	Tip Size	Length			
		Tip Size	21 mm	25 mm	31 mm
	14	04c	5723943	5723944	5723945

For Calcified Canals:

For Narrow Canals:

For Medium Canals:

For Wide Canals:

6 - Pack STERILE	Tip Size	Tip Size	Length		
			21 mm	25 mm	31 mm
	20	08v	5723940	5723941	5723942
	25	08v	9885384	9885385	9885386
	40	06v	9885387	9885388	9885389
	50	05v	9885390	9885391	9885392
Assorted Pack			9885393	9885394	9885395



Case was crafted using R25

**Professor Gianluca Gambarini,**  
University of Rome, La Sapienza School, Rome, Italy




*"I absolutely love this product, I love that I didn't break any files, and I love the ease of use and feel of the files."*

Dr. Gottlieb, DMD, Mount Laurel, NJ

*"The system allowed for simple, fast, efficient prepping of root canals, especially on curved roots."*

Dr. Dela Cruz

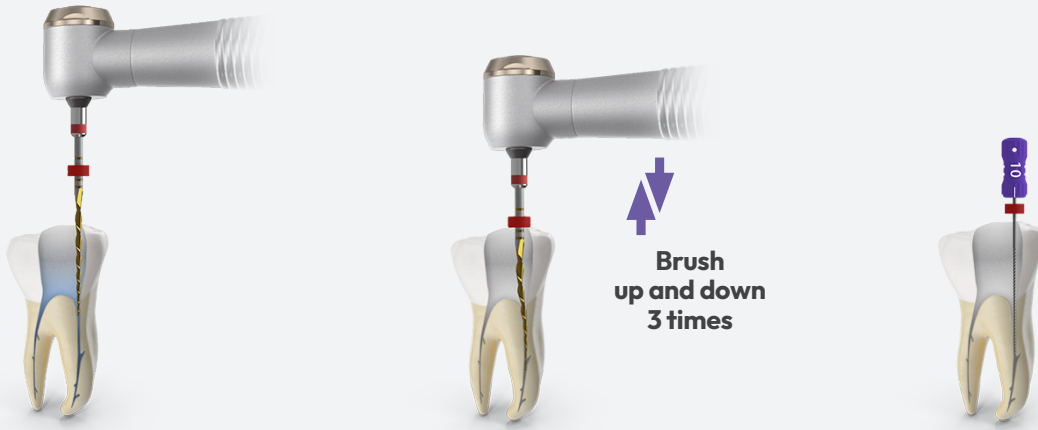

## Root Canal Shaping

Confirm straight line access to all orifices.

You may treat most canals with the R25. Please refer to full Instruction For Use.

- ① Use preoperative radiographs to estimate the working length.
- ② Irrigate and fill the chamber with solution.
- ③ Insert the **EdgeOne R** instrument to the orifice and start the motor in Reciprocation mode.
- ④ With light pressure, use 3 slow pecking motions of not greater than 3 mm and advance the instrument down the canal.
- ⑤ Irrigate the canal and clear the canal of debris with an ISO 10 instrument to the estimated working length.
- ⑥ Repeat Steps 3-5 until the coronal 2/3 is shaped.
- ⑦ Electronically determine the working length with a hand instrument ISO size 10.
- ⑧ Repeat steps 3-5 with **EdgeOne R** until reaching the working length.



PRODUCTS

## Gutta Percha & Paper Points



### EdgeOne R Utopia™ Gutta Percha Points 60/ Pack

Tip Size	Item Code
<b>R20</b>	5722119
<b>R25</b>	5722120
<b>R40</b>	5722123
<b>R50</b>	5722124
Assorted Pack*	5722125

\* composition on the assorted pack:  
R20x20, R25x20, R40x10 and R50x10

### EdgeEndo® Paper Points 60/ Pack

Tip Size	Item Code
<b>20</b>	5722091
<b>25</b>	5722095
<b>40</b>	5722101
<b>50</b>	5722105
Assorted Pack*	5722107

\* composition on the assorted pack:  
20x10, 25x10, 30x10, 35x10, 40x10 and 45x10



**FOLLOW US ON  
SOCIAL MEDIA**