



MESI

Simplifying Diagnostics

MESI mTABLET ABI & MESI mTABLET TBI

Complete arterial assessment in two simple steps

- Wireless, simultaneous **Ankle-Brachial Index measurement in 1 minute**
- **1-step, automated Toe-Brachial Index** extension for immediate further PAD assessment
- ABI and TBI measurement **reports are automatically synced** with the built-in electronic medical record
- Access to the **full history of previous measurements** and automatic **patient trend analysis**
- Immediate **specialist consultation** through the safe share option
- **Safe storage** of medical data



Arterial assessment in two steps

STEP 1: ABI measurement

Measure ABI with MESI mTABLET ABI on all patients in the cardiovascular risk group.

1.41 or more non-compressible	1.40 - 1.00 normal	0.99 - 0.91 borderline	0.90 - 0.51 abnormal	0.50 or less severe
---	------------------------------	----------------------------------	--------------------------------	-------------------------------

STEP 2: TBI measurement

Measure TBI after MESI mTABLET ABI displays a PAD message, an ABI result above 1.41 and on all patients with diabetes, chronic kidney disease, metabolic syndrome or old age.

above 0.70 normal result	below 0.70 possible arterial condition
------------------------------------	--

Why MESI mTABLET ABI


- SmartArm™ detection – to determine the higher blood pressure of the two
- Automated, 3-cuff simultaneous measurement
- Multiple cuff sizes and ability to mix-and-match different size cuffs during one measurement
- Pulse waveforms and oscillation graphs
- Advanced review and alerts, thanks to PADsense™ algorithm



Why MESI mTABLET TBI

- Safe, simultaneous measurements in both arms and toes, with adaptive, infrared LED PPG light, detecting toe skin temperature and thickness
- PADsense™ algorithm for detection of severe Peripheral Arterial Disease
- Comprehensive and reliable TBI report with pulse waveforms and oscillation graphs for the arms and PPG pulse waveforms for the toes
- Availability of single-use (disposable) toe cuffs

Expand the use of your ABI and TBI devices through smart applications

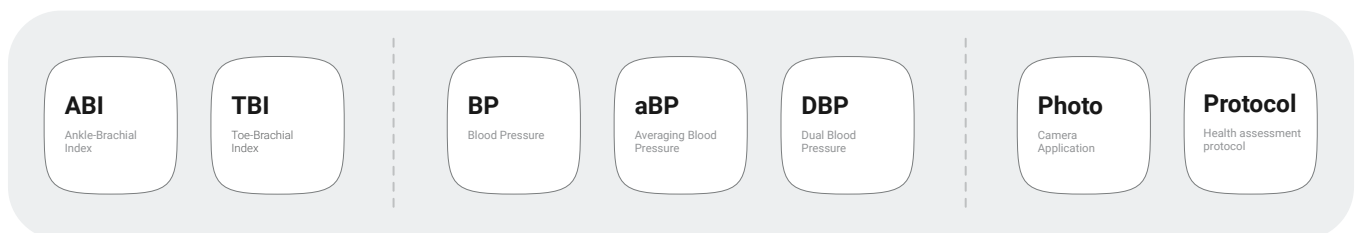

MESI, Ltd.
Leskoškova cesta 11a
1000 Ljubljana, Slovenia, EU
E: info@mesimedical.com
T: +386 1 620 34 87

www.mesimedical.com
Facebook / MESIdoo
Twitter / MESImedical
LinkedIn / MESI

No.1 Leading provider of Ankle-Brachial Index measuring technology
Worldwide presence
European production and development

SIQ
ISO 9001 Q-1664
ISO 13485 M-049

CERTIFIED
Net
MANAGEMENT SYSTEM

CE 1304, ISO 9001 and ISO 13485 certified

CE 1304